ClinChek® - Control

Urine Control for Catecholamines (FD) lyophilised / Kontrollurin für Katecholamine (FD) lyophilisiert

Intended use:

ClinChek® Urine Controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human urine and are available with mean values in the normal as well as in the pathological range. After reconstitution these controls have to be treated like patient samples in one series of analyses and are intended to be used with the ClinRep® Complete Kit for Catecholamines in urine (On-Line Analysis) with fluorescence detection (FD), order no. 2100.

Reconstitution:

Add exactly 5.0 ml of 0.1 M HCl to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised urine control is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- at least 12 hours when stored at 15 - 30 °C
- at least 5 days when stored at 2 - 8 °C
- at least 30 days when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® Controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 0.66 %.

Mean values:

The mean values and confidence intervals have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK) with statistical methods.

Pack size:

ClinChek® Urine Control
Level I
10 x 5 ml, **order no.: 2180**
Level II
10 x 5 ml, **order no.: 2181**
Level I, II
2 x 5 x 5 ml, **order no.: 2182**

Precautions:

These controls were manufactured from human urine. Therefore the material has to be considered as potentially infectious and should be treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH