ClinCal® - Calibrator

Serum Calibrator lyophilised, for Vitamin B6 /

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The mean values have been assigned with the ClinRep® Complete Kit for Vitamin B6. The uncertainty of the mean values is $\pm 0.8\%$ for both analytes at a confidence level of 95 %. The mean value of PLP is traceable to NIST-SRM3950.

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Intended use:

ClinCal® Serum Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 3.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C in the dark, the lyophilised serum calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is when stored in the dark:

- at least 12 hours when stored at 15 - 30 °C
- at least 4 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C
(avoid repeated freezing and thawing)

The variation of the filling volume (CV) is $< 1\%$. The average residual moisture of this lot is 1.22 %.

Pack size:

ClinCal® Serum Calibrator for Vitamin B6: 1 x 3 ml, **order no.: 26013**

Caution:

The human serum which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen, HIV1- and HCV-RNA (NAT). Nevertheless, the serum calibrator should be considered as potentially infectious and treated with appropriate care.

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