ClinChek® - Control Plasma Control lyophilised / Kontrollplasma lyophilisiert

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Intended use:

ClinChek® plasma controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human plasma and are available with mean values in the therapeutic as well as in the elevated range. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 3.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised plasma control is stable for 48 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- at least 12 hours when stored at 15 - 30 °C
- at least 7 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.30 %.

Mean values:

The mean values and confidence intervals have been established at RECIPE, according to the Guideline of the German Medical Association for Quality Assurance (Rili-BÄK), with statistical methods.

Pack size:

ClinChek® Plasma Control
Level I
10 x 3 ml, **order no.: 15180**
Level II
10 x 3 ml, **order no.: 15181**
Level I, II
2 x 5 x 3 ml, **order no.: 15182**

Precautions:

The human plasma which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, hepatitis B-surface antigen, HIV1- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the plasma controls should be considered as potentially infectious and treated with appropriate care.

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Technique of analysis / Analysentechnik: RECIPE - HPLC

ClinChek® - Control

Plasma Control, Level II

Kontrollplasma, Level II

REF	15181
LOT	512
	2018-03

Analyte / Analyt	Unit / Einheit	Mean Value / Sollwert	Control Range / Kontrollbereich	Unit / Einheit	Mean Value / Sollwert	Control Range / Kontrollbereich
Sulthiame / Sultiam	mg/l	14.6	11.7 - 17.5	µmol/l	50.3	40.3 - 60.3
Lamotrigine / Lamotrigin	mg/l	13.7	11.0 - 16.4	µmol/l	53.5	43.0 - 64.0

Technique of analysis / Analysentechnik: RECIPE - HPLC

RECIPE CHEMICALS + INSTRUMENTS GmbH
Dessauerstr. 3 D-80992 Munich / Germany
Tel.: +49 / 89 / 54 70 81 - 0 Fax: +49 / 89 / 54 70 81 - 11
Internet: www.recipe.de e-mail: info@recipe.de