ClinCal® - Calibrator

Urine Calibrator lyophilised, for Catecholamines (FD) /

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Intended use:

ClinCal® Urine Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be treated like a patient sample. This calibrator is intended to be used with the ClinRep® Complete Kit for Catecholamines (On-line Analysis) with fluorescence detection (FD), order no. 2100.

Reconstitution:

Add exactly 8.0 ml of 0.1 M HCl to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised urine calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- at least 12 hours when stored at 15 - 30 °C
- at least 5 days when stored at 2 - 8 °C
- at least 30 days when stored below -18 °C
(avoid repeated freezing and thawing)

The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 0.47 %.

Pack size:

ClinCal® Urine Calibrator for Catecholamines (FD): 5 x 8 ml, **order no.: 2113**

Caution:

This calibrator was manufactured from human urine. Therefore the material has to be considered as potentially infectious and should be treated with appropriate care.

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