ClinChek® - Control Plasma Control lyophilised / Kontrollplasma lyophilisiert

FOR ANTIMYCOTICS / FÜR ANTIMYKOTIKA

Intended use:

ClinChek® plasma controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human plasma and are available with mean values in the therapeutic as well as in the elevated range. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 1.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised plasma control is stable for 48 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 8 hours when stored at 15 - 30 °C
- at least 7 days when stored at 2 - 8 °C
- at least 30 days when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.45 %.

Mean values:

The mean values and confidence intervals have been established in independent reference laboratories with supervision of RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK) with statistical methods.

Pack size:

ClinChek® Plasma Control
Level I, II, III
3 x 2 x 1 ml, **order no.: 8839**

Precautions:

The human plasma which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV1- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the plasma controls should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH