ClinChek® - Control
Serum Control lyophilised / Kontrollserum lyophilisiert

REF	8851
LOT	449
	2017-12

DRUGFREE

Intended use:

ClinChek® serum controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human serum. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 5.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

If it is necessary to have a defined bufferconstitution of the control for further sample preparation (e.g. for pH-adjustment), you can also dissolve the lyophilised serum control directly in the appropriate buffer.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised serum control is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the serum is:

- at least 5 hours when stored at 15 - 30 °C
- at least 24 hours when stored at 2 - 8 °C
- at least 10 days when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.19 %.

Pack size:

ClinChek® Serum Control
 10 x 5 ml, **order no.: 8851**

Precautions:

The human serum which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV1- and HCV-RNA (NAT). Nevertheless, the serum controls should be considered as potentially infectious and treated with appropriate care.

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