ClinCal® - Calibrator

Serum Calibrator lyophilised / Serum-Kalibrator lyophilisiert

FOR PSYCHOACTIVE DRUGS TDM4 / FÜR PSYCHOPHARMAKA TDM4

Intended use:

ClinCal® serum calibrators are used for calibration of your analysis-system. This lyophilised calibrator is based on human serum and is intended for a single-point calibration. After reconstitution the calibrator has to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 5.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised serum calibrator is stable for 36 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 3 days when stored at 15 - 30 °C
- at least 7 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinCal® calibrators have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 0.94 %.

Mean values:

The mean values have been established in independent reference laboratories with supervision of RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods. For value assignment chromatographic methods were used.

Pack size:

ClinCal® Serum Calibrator
5 x 5 ml, **order no.: 9711**

Precautions:

The human serum which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV- and HCV-RNA, HBV- DNA (NAT). Nevertheless, the serum calibrator should be considered as potentially infectious and treated with appropriate care.

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Technique of Analysis / Analysentechnik:
 HPLC-UV, LC-MS/MS

RECIPE CHEMICALS + INSTRUMENTS GmbH
 Dessauerstr. 3 D-80992 Munich / Germany
 Tel.: +49 / 89 / 54 70 81 - 0 Fax: +49 / 89 / 54 70 81 - 11
 Internet: www.recipe.de e-mail: info@recipe.de