ClinCal® - Calibrator

Plasma Calibrator lyophilised / Plasma-Kalibrator lyophilisiert

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Intended use:

ClinCal® plasma calibrators are used for calibration of your analysis-system. This lyophilised calibrator is based on human plasma and is intended for a single-point calibration. After reconstitution, the calibrator has to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 3.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

If it is necessary to have a defined buffer-constitution of the control for further sample preparation (e.g. for pH-adjustment), you can also dissolve the lyophilised plasma calibrator directly in the appropriate buffer.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised plasma calibrator is stable for 36 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 5 hours when stored at 15 - 30 °C
- at least 24 hours when stored at 2 - 8 °C
- at least 10 days when stored below -18 °C (avoid repeatedly freezing and thawing)

Notes:

Routine analyses of the plasma calibrator are performed according to the testprocedure for patient samples. The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinCal® calibrators have to pass strict qualitycontrol procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 1.04 %.

Mean values:

The mean values were established in independent reference laboratories with supervision of RECIPE, according to the guideline of the German Medical Council for quality assurance (RiliBäk). For value assignment aminoacid analyzers (AAA) were used.

Pack size:

ClinCal® Plasma Calibrator
5 x 3 ml, **order no.: 10213**

Precautions:

The human plasma that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the plasma calibrator should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH