ClinCal® - Calibrator

Serum Calibrator lyophilised / Serum-Kalibrator lyophilisiert

FOR ANTIPSYCHOTIC DRUGS TDM3 / FÜR PSYCHOPHARMAKA TDM3

Intended use:

ClinCal® serum calibrators are used for calibration of your analysis-system. This lyophilised calibrator is based on human serum and is intended for a single-point calibration. After reconstitution, the calibrator has to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 5.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

If it is necessary to have a defined buffer-constitution of the control for further sample preparation (e.g. for pH-adjustment), you can also dissolve the lyophilised serum calibrator directly in the appropriate buffer.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised serum calibrator is stable for 36 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 5 hours when stored at 15 - 30 °C
- at least 1 day when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C (avoid repeatedly freezing and thawing)

Notes:

Routine analyses of the serum calibrator are performed according to the testprocedure for patient samples. The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinCal® calibrators have to pass strict qualitycontrol procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 1.26 %.

Mean values:

The mean values were established in independent reference laboratories with supervision of an institution that is recommended by the guideline of the German Medical Council for quality assurance (RiLiBäk). For value assignment chromatographic methods were used.

Pack size:

ClinCal® Serum Calibrator
5 x 5 ml, **order no.: 9708**

Precautions:

The human serum that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2-, HbC- and HCV-antibodies, hepatitis B-surface antigen, HIV-, HBV- and HCV-genome (PCR). Nevertheless, the serum calibrator should be considered as potentially infectious and treated with appropriate care.

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HPLC-UV, LC-MS/MSRECIPE CHEMICALS + INSTRUMENTS GmbH