**ClinTest<sup>®</sup> - Standard****Test Solution for Risperidone, Paliperidone /****Testlösung für Risperidon, Paliperidon**

for HPLC Assay in Plasma/Serum

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**Intended use:**

ClinTest<sup>®</sup> Test Solutions are intended to check the daily performance of your HPLC - system. This test solution needs no preparation and can be injected directly into your analysing system (injection volume: 750 µl). After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the test solution the first time, you can compare your results with the chromatogram on the backside of this data sheet. Besides retention times you are also able to check peak shape, integration and baseline with your test chromatogram.

**Storage and stability:**

Originally closed and stored at 2 - 8 °C, the ready -to-use standard solution is stable for 36 months, but not longer than the expiration date printed on the label. After opening the vial, the standard solution is stable for at least 4 weeks, when stored at 2 - 8 °C.

**Pack size:**ClinTest<sup>®</sup> Test Solution for Risperidone, Paliperidone: 15 ml, **order no.: 40114****Anwendung:**

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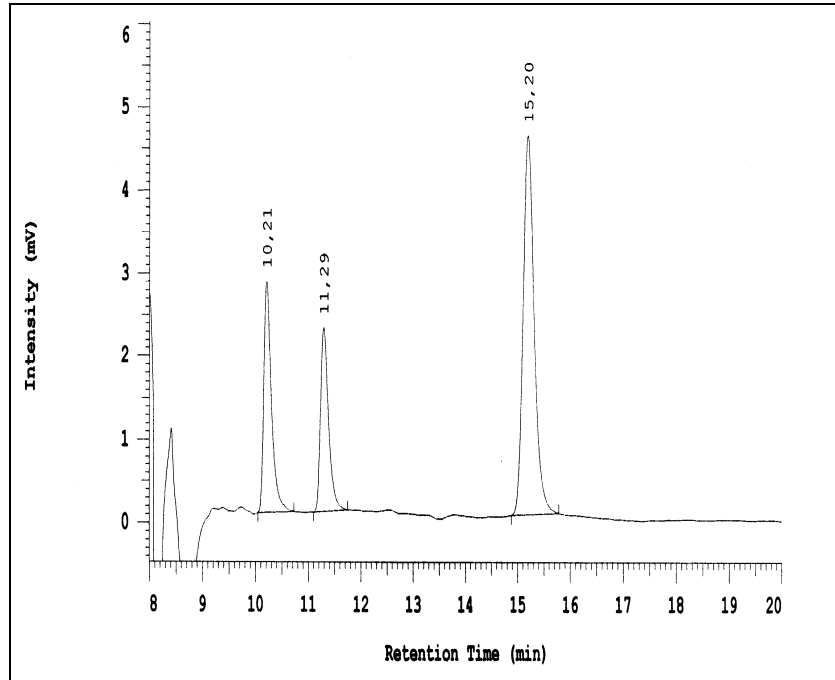
**Please note:** The test solution (order no. 40114) does not contain the Internal Standard (order no. 40012). For the chromatogram above, there have been added 10 µl of the Internal Standard Solution per 1 ml of the Test Solution.

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RECIPE                      CHEMICALS + INSTRUMENTS GmbH