ClinTest[®] - Standard**Hippuric Acids Standard Solution/****Hippursäuren Standardlösung**

for HPLC Assay in Urine/

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REF	52011
LOT	818
	2012-05

Constituent / Bestandteil	Concentration / Konzentration
Hippuric Acid / Hippursäure	25 mg/l (140 µmol/l)
o-Methyl-Hippuric Acid / o-Methyl-Hippursäure	10 mg/l (52 µmol/l)
Internal Standard / Interner Standard	10 mg/l (72 µmol/l)
m-Methyl-Hippuric Acid / m-Methyl-Hippursäure	10 mg/l (52 µmol/l)

Intended use:

ClinTest[®] Standard Solutions are intended to check the daily performance of your HPLC - system. This standard solution needs no preparation and can be injected directly into your analysing system (injection volume: 20 µl). After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the standard solution the first time, you can compare your results with the chromatogram on the backside of this data sheet. Besides retention times you are also able to check peak shape, integration and baseline with your standard chromatogram.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the ready -to-use standard solution is stable for 24 months, but not longer than the expiration date printed on the label. After opening the vial, the standard solution is stable for at least 4 weeks, when stored at 2 - 8 °C.

Pack size:ClinTest[®] Standard Solution for Hippuric Acids in Urine:1 x 3 ml, **order no.: 52011****Anwendung:**

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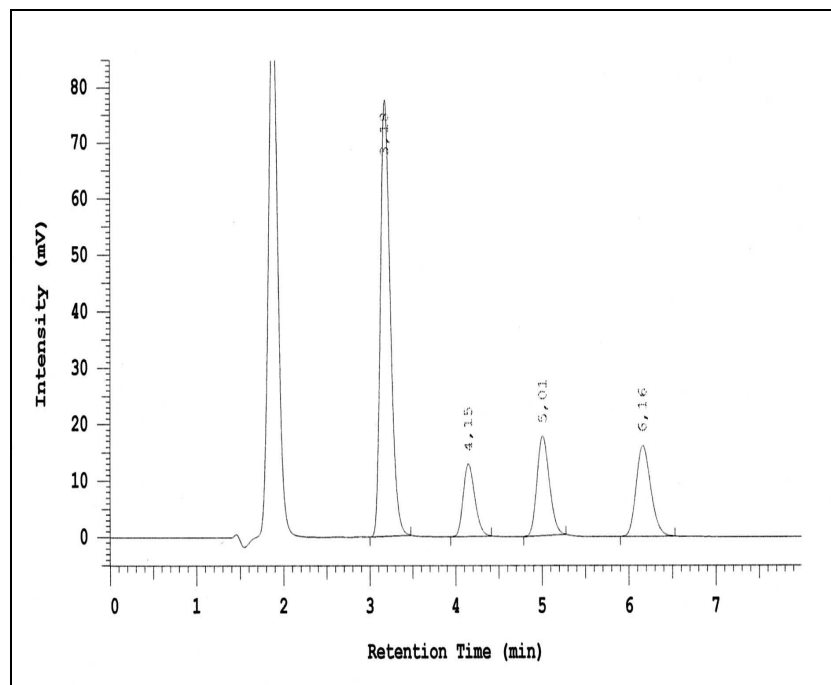
Hippuric Acid / Hippursäure	3.18 min
o-Methyl-Hippuric Acid / o-Methyl-Hippursäure	4.15 min
Internal Standard / Interner Standard	5.01 min
m-Methyl-Hippuric Acid / m-Methyl-Hippursäure	6.16 min