ClinCal[®] - Calibrator

Urine Calibrator lyophilised / Urin-Kalibrator lyophilisiert

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Intended use:

ClinCal[®] urine calibrators are used for calibration of your analyzing system in the occupational and environmental toxicological laboratory as well as in the clinical-chemical one. This lyophilised calibrator is based on human urine and is available in a range of concentration, that allows a comfortable one-point-calibration. After reconstitution, the calibrator has to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 10.0 ml of analytical grade or bidistilled water to the vial and close it well. Then dissolve the lyophilised material completely. For this purpose we recommend a so-called roller mixer; a treatment of 30 min with such an instrument is sufficient. Otherwise incubate the samples for 1 to 2 hours at room temperature by shaking it carefully from time to time. Prior to use mix the calibrator well again.

Storage and stability:

Originally closed and stored at 4 °C the lyophilised urine calibrator is stable for 4 years, but not longer than the expiration date printed on

the label. After reconstitution the stability of the analytes is:

- at least 8 hours when stored at + 20 °C
- at least 7 days when stored at + 4 °C
- at least 1 month when stored at - 20 °C (avoid repeatedly freezing and thawing)

If you intend to use a reconstituted calibrator for the determination of mercury, zinc and aluminium you will have a decrease of the value or contamination. Therefore we recommend to use a freshly dissolved calibrator for these analytes.

Notes:

Routine analyses of the calibrator are performed according to the testprocedure for patient samples. The concentration of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinCal[®] calibrators have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 1.04 %.

Mean values:

The mean values were established in independent reference laboratories with supervision of an institution, that is recommended by the rules of the German Bundesärztekammer (RiLiBäk). For the determination of the trace elements, atomic absorption spectrometry (AAS) was used. Even if you use different techniques of analysis, you should find your control results within the given ranges of the data sheet, after having calibrated your analyzing system with this ClinCal[®] urine calibrator.

Pack size:

ClinCal[®] Urine Calibrator
5 x 10 ml, **order no.: 9988**

Precautions:

The calibrator was manufactured from human urine. Therefore the material has to be considered as potentially infectious and should be treated with appropriate care.

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CHEMICALS + INSTRUMENTS GmbH Labortechnik