## ClinCal® - Calibrator

Urine Calibrator Set lyophilised / Urin-Kalibrator-Set lyophilisiert

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### Intended use:

ClinCal® urine calibrators are used for calibration of your analyzing system. These lyophilised calibrators are based on human urine and are available as a calibrator set with 7 levels, including a blank. The calibrators have to be prepared like patient samples in one series of analyses.

### Reconstitution:

Add exactly 3.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

### Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised urine calibrators are stable for 36 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 12 hours when stored at 15 - 30 °C
- at least 7 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C (avoid repeatedly freezing and thawing)

### Notes:

The mean values of the analytes are chosen in a way that a linear calibration curve can be generated over a wide range of concentration. According to quality assurance all ClinCal® calibrators have to pass strict qualitycontrol procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 0.51 %.

Creatinine concentrations of the different calibrator levels are between 0.5 g/l and 1.50 g/l.

### Mean values:

The mean values were established in additional independent reference laboratories with supervision of RECIPE, according to the guideline of the German Medical Council (RiliBäk) with statistic methods.

### Pack size:

ClinCal® Urine Calibrator Set for Nicotine and -Metabolites Level 0 - 6  
7 x 1 x 3 ml, **order no.: MS3013**

### Precautions:

These calibrators were manufactured from human urine. Therefore the material has to be considered as potentially infectious and should be treated with appropriate care.

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RECIPE                      CHEMICALS + INSTRUMENTS GmbH