## ClinCal<sup>®</sup> - Calibrator

Whole Blood-Calibrator lyophilised, for Vitamin B1, B2, B6

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### Intended use:

ClinCal<sup>®</sup> Whole Blood-Calibrators are used for calibration of the HPLC-system. This lyophilised calibrator has to be prepared like a patient sample.

### Reconstitution:

Add exactly 2.0 ml HPLC-water to the vial. To dissolve the material completely, we recommend a so called roller mixer. A treatment of 30 to 60 minutes with such an instrument is sufficient.

### Storage and stability:

Originally closed and stored at 2 - 8 °C in the dark, the lyophilised whole blood-calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is when stored in the dark:

- at least 5 hours when stored at 15 - 30 °C
- at least 24 hours when stored at 2 - 8 °C
- at least 10 days when stored below -18 °C  
(avoid repeatedly freezing and thawing)

The average residual moisture of this batch is 0.78 %.

### Pack size:

ClinCal<sup>®</sup> Whole Blood-Calibrator for Vitamin B1, B2, B6: 5 x 2 ml, **order no.: 24513**

### Caution:

The human blood that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the whole blood-calibrator should be considered as potentially infectious and treated with appropriate care.

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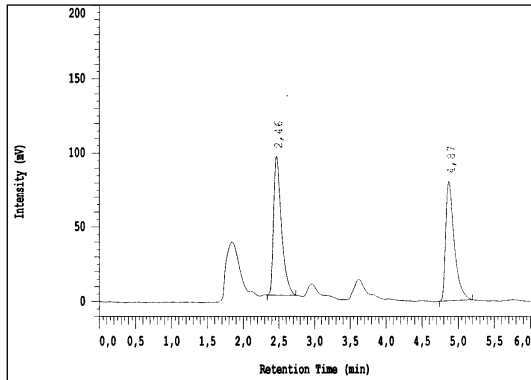
### Vitamin B1 - Chromatography / Chromatographie:

Without enzymatic treatment (free Thiamine) /  
ohne Enzymbehandlung (freies Thiamin):

Thiaminepyrophosphate (TPP) /  
Thiaminpyrophosphat (TPP)            2.46 min  
Thiamine / Thiamin                        4.87 min

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RECIPE                            CHEMICALS + INSTRUMENTS GmbH