ClinCal[®] - Calibrator

Whole Blood Calibrator, lyophil., for Immunosuppressants

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Intended use:

ClinCal[®] Whole Blood Calibrators are used for calibration of the analysis-system. This lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 2.0 ml HPLC-water to the vial. To dissolve the material completely, we recommend a so called roller mixer. A treatment of 30 to 60 minutes with such an instrument is sufficient.

Storage and stability:

Originally closed and stored at 2 - 8 °C in the dark, the lyophilised whole blood calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- at least 8 hours when stored in the dark at 15 - 30 °C
- at least 7 days when stored in the dark at 2 - 8 °C
- at least 30 days when stored in the dark below -18 °C
(avoid repeatedly freezing and thawing)

The average residual moisture of this lot is 0.50 %.

Pack size:

ClinCal[®] Whole Blood Calibrator for Immunosuppressants: 5 x 2 ml, **order no.: 5013**

Caution:

The human whole blood that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the calibrator should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH