## ClinCal<sup>®</sup> - Calibrator

Whole Blood Calibrator Set lyophilised / Vollblut-Kalibrator-Set lyophilisiert

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### Intended use:

ClinCal<sup>®</sup> whole blood calibrators are used for calibration of the analysis-system. These lyophilised calibrators are based on human blood and are available as a calibrator set with 7 levels, including a blank. The calibrators have to be prepared like patient samples in one series of analyses.

### Reconstitution:

Add exactly 2.0 ml of HPLC-water to the vial. To dissolve the material completely, we recommend a so-called roller mixer. A treatment of 30 to 60 minutes on such an instrument is sufficient.

### Storage and stability:

Originally closed and stored at 2 - 8 °C in the dark, the lyophilised whole blood calibrators are stable for 36 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 8 hours when stored at 15 - 30 °C in the dark
- at least 7 days when stored at 2 - 8 °C in the dark
- at least 30 days when stored below -18 °C in the dark (avoid repeatedly freezing and thawing)

### Notes:

The mean values of the analytes are chosen in a way that a linear calibration curve can be generated over a wide range of concentration. According to quality assurance all ClinCal<sup>®</sup> calibrators have to pass strict qualitycontrol procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 0.93 %

### Mean values:

The mean values were established in additional independent reference laboratories with supervision of RECIPE, according to the guideline of the German Medical Council (RiliBäk) with statistic methods.

### Pack size:

ClinCal<sup>®</sup> Whole Blood Calibrator Set for Immunosuppressants  
Level 0 - 6  
7 x 1 x 2 ml, **order no.: 9933**

### Precautions:

The human whole blood that was used for manufacturing the calibrators was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the calibrators should be considered as potentially infectious and treated with appropriate care.

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2013-07

**Level 0**

This blank-level is manufactured from the same pool of human blood like the others, but it was not spiked with any immunosuppressant. Therefore the concentration of each of the four analytes is zero.

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RECIPE CHEMICALS + INSTRUMENTS GmbH