## ClinChek® - Control

### Whole Blood Control lyophilised / Vollblut-Kontrolle lyophilisiert

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##### Intended use:

ClinChek® whole blood controls are used for internal quality assurance in the occupational and environmental toxicological laboratory as well as in the clinical-chemical one. These lyophilised controls are based on human whole blood and are available in three different ranges of concentration. After reconstitution, the controls have to be prepared like patient samples in one series of analyses.

##### Reconstitution:

Add exactly 5.0 ml of analytical grade or bidistilled water to the vial and close it well. Then dissolve the lyophilised material completely. For this purpose we recommend a so-called roller mixer; a treatment of 60 min with such an instrument is sufficient. Otherwise incubate the samples for 2 hours at room temperature by shaking it carefully from time to time. Prior to use mix the controls well again.

##### Storage and stability:

Originally closed and stored at 2 - 8 °C the lyophilised whole blood control is stable for 48 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 8 hours when stored at 15 - 30 °C
- at least 4 days when stored at 2 - 8 °C
- at least 1 month when stored below -18 °C (avoid repeatedly freezing and thawing)

If performing mercury analyses, use only freshly dissolved material.

##### Notes:

Routine analyses of the controls are performed according to the testprocedure for patient samples. The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIFE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 0.76 %.

##### Mean values:

The mean values and confidence intervals were established in independent reference laboratories with supervision of RECIFE. The calculation of the mean values and confidence intervals was done according to the guideline of the German Medical Council (RiliBäk) with statistic methods. For the determination of the metals, atomic absorption spectrometry (AAS) was used as well as ICP-MS. Even if you use different techniques of analysis your results should be within the given ranges of the data sheet.

##### Pack size:

ClinChek® Whole Blood Control  
Level I  
10 x 5 ml, **order no.: 8840**  
Level II  
10 x 5 ml, **order no.: 8841**  
Level III  
10 x 5 ml, **order no.: 8842**  
Level I, II and III  
3 x 3 x 5 ml, **order no.: 8843**

##### Precautions:

The human whole blood that was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV 1/2-, HBV- and HCV-antibodies, hepatitis B-surface antigen, HIV-, HBV- and HCV-genome (PCR). Nevertheless, the controls should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH