ClinTest® - Standard

Test Solution for CDT
for HPLC Assay in Serum

REF	21014
LOT	1039
	2021-01

Intended use:

ClinTest® Test Solutions are intended to check the daily performance of your HPLC - system. This Test Solution needs no preparation and can be injected directly into your analysing system (injection volume: 400 µl). In case of any turbidity, please centrifuge before injection. After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the Test Solution for the first time, you can compare your results with the chromatogram below. Besides retention times you are also able to check peak shape, integration and baseline with your standard chromatogram.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C. After opening the vial, the ready-to-use Test Solution is stable for 4 weeks, when stored at 2 - 8 °C.

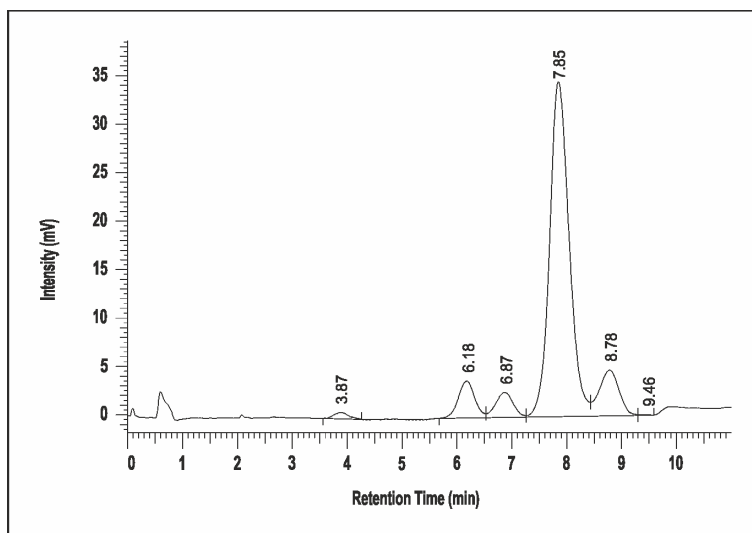
Pack size:

ClinTest® Test Solution for CDT: 1 x 15 ml, order no.: 21014

Precautions:

The human serum which was used for manufacturing the Test Solution was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies as well as Hepatitis B-surface antigen. Nevertheless, the test solution should be considered as potentially infectious and treated with appropriate care.

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RECIFE CHEMICALS + INSTRUMENTS GmbH
RECIPE CHEMICALS + INSTRUMENTS GmbH