ClinCal® - Calibrator

Whole Blood Calibrator lyophilised, for Vitamin B1, B2, B6 /

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Intended use:

ClinCal® Whole Blood Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 2.0 ml HPLC-water to the vial. To dissolve the material completely, we recommend a so-called roller mixer. A treatment of 30 to 60 minutes with such an instrument is sufficient.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C in the dark. After reconstitution the stability of the analytes is, when stored in the dark:

- 5 hours at 15 - 30 °C
- 1 day at 2 - 8 °C
- 10 days at below -18 °C (avoid repeated freezing and thawing)

The variation of the filling volume (CV) is < 1 %. The average residual moisture of this batch is 0.71 %.

Pack size:

ClinCal® Whole Blood Calibrator for Vitamin B1, B2, B6: 5 x 2 ml, **order no.: 24513**

Caution:

The human whole blood which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV1- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the whole blood calibrator should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH