ClinChek® - Control
Plasma Control lyophilised / Kontrollplasma lyophilisiert
FOR ATYPICAL NEUROLEPTICS / FÜR ATYPISCHE NEUROLEPTIKA

REF	40080 - 40082
LOT	1476
	2020-11

Intended use:

ClinChek® Plasma Controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human plasma and are available with mean values that cover the therapeutic range. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 3.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C. After reconstitution the stability of the analytes is:

- 1 day at 15 - 30 °C
- 7 days at 2 - 8 °C
- 90 days at below -18 °C (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® Controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.15 %.

Mean values:

The mean values and confidence intervals have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Pack size:

ClinChek® Plasma Control
 Level I, II
 2 x 5 x 3 ml, **order no.: 40082**

Precautions:

The human plasma which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen. Nevertheless, the plasma controls should be considered as potentially infectious and treated with appropriate care.

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