ClinChek® - Control

Hemoglobin Control (low) lyophilised / Hämoglobinkontrolle (low) lyophilisiert

Intended use:

ClinChek® Controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human erythrocytes and are available with mean values in three different ranges of concentration. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 1.0 ml of reconstitution solution to the vial.

Caution:

The reconstitution solution, which is supplied with the controls, must be adapted to room temperature before use and can then be stored also at ambient temperature.

To dissolve the material completely, we recommend a so-called roller mixer. A treatment of 15 minutes on such an instrument is sufficient. In addition, we recommend the use of a vortex mixer.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C.

After reconstitution the stability of the analyte is:

- 1 day at 15 - 30 °C
 - 2 days at 2 - 8 °C
 - 90 days below -18 °C
- (avoid repeated freezing and thawing)

Notes:

The concentrations of the analyte are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® Controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %.

Mean values:

The mean values and confidence intervals have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Pack size:

ClinChek® Control
 Level I (incl. reconstitution solution)
 9 x 1 ml, **order no.: 8912**
 Level II (incl. reconstitution solution)
 9 x 1 ml, **order no.: 8913**
 Level III (incl. reconstitution solution)
 9 x 1 ml, **order no.: 8914**
 Level I, II, III (incl. reconstitution solution)
 3 x 3 x 1 ml, **order no.: 8916**

Precautions:

The human erythrocytes which were used for manufacturing the controls were tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV1- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the controls should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH