ClinTest® - Standard

Standard Solution for Lamotrigine, Sulthiame /

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Internal Standard 1:

Relevant for use in combination with ClinRep® Complete Kit for Lamotrigine and Sulthiame, order no.: 15100.

Internal Standard 2:

Relevant for use in combination with ClinRep® Complete Kit for Antiepileptics, order no.: 15000.

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Intended use:

ClinTest® Standard Solutions are intended to check the daily performance of your HPLC - system. This Standard Solution needs no preparation and can be injected directly into your analysing system (injection volume: 20 µl or 10 µl if you use the ClinRep® Complete Kit for Antiepileptics, order no. 15000, respectively). After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the Standard Solution for the first time, you can compare your results with the chromatogram on the back of this data sheet. Besides retention times you are also able to check peak shape, integration and baseline with your standard chromatogram. For calibration the ClinCal® Plasma Calibrator, order no. 15113, has to be used.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the ready-to-use Standard Solution is stable for 36 months, but not longer than the expiration date printed on the label. After opening the vial, the standard solution is stable for at least 4 weeks, when stored at 2 - 8 °C.

Pack size:

ClinTest® Standard Solution for Lamotrigine, Sulthiame: 1 x 3 ml, **order no.: 15111**

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