ClinChek® - Control Serum Control lyophilised / Kontrollserum lyophilisiert

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Intended use:

ClinChek® Serum Controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human serum and are available with mean values in two different ranges of concentration. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 0.5 ml of HPLC-water to the vial and mix for max. 3 min. After that the lyophilisate is reconstituted and ready to use.

Caution: Immediately after reconstitution the serum controls have to be prepared according to the instruction manual of the ClinRep® Complete Kit for Vitamin C and analysed.

Storage and stability:

Originally closed and stored below -18 °C in the dark, the lyophilised serum control is stable for 24 months, but not longer than the expiration date printed on the label. After reconstitution the serum controls have to be used immediately and can not be stored.

Notes:

The concentrations of the analyte are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® Controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.31 %.

Mean values:

The mean values and confidence intervals have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Pack size:

ClinChek® Serum Control:
Level I
10 x 0.5 ml, **order no.: 28080**
Level II
10 x 0.5 ml, **order no.: 28081**
Level I, II
2 x 5 x 0.5 ml, **order no.: 28082**

Precautions:

The human serum which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen, HIV- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the serum controls should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH