ClinChek® - Control Serum Control lyophilised / Kontrollserum lyophilisiert

FOR VITAMINS / FÜR VITAMINE

Intended use:

ClinChek® Serum Controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human serum and are available with mean values in the normal as well as in the pathological range. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 5.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored below -18 °C in the dark, the lyophilised serum control is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- 8 hours
when stored at 15 - 30 °C in the dark
- 3 days
when stored at 2 - 8 °C in the dark
- 3 months
when stored below -18 °C in the dark (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® Controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.08 %.

Mean values:

The mean values and confidence intervals have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Pack size:

ClinChek® Serum Control
Level I
10 x 5 ml, **order no.: 8870**
Level II
10 x 5 ml, **order no.: 8871**
Level III
10 x 5 ml, **order no.: 8872**
Level I, II, III
3 x 3 x 5 ml, **order no.: 8873**

Precautions:

The human serum which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen, HIV- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the serum controls should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH