ClinCal[®] - Calibrator

Serum Calibrator lyophilised, for Levetiracetam (Keppra[®]) /

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Intended use:

ClinCal[®] Serum Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 3.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised serum calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analyte is:

- at least 12 hours when stored at 15 - 30 °C
- at least 7 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C
(avoid repeated freezing and thawing)

The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.02 %.

Pack size:

ClinCal[®] Serum Calibrator for Levetiracetam: 1 x 3 ml, **order no.: 15513**

Caution:

The human serum which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV1- and HCV-RNA (NAT). Nevertheless, the serum calibrator should be considered as potentially infectious and treated with appropriate care.

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