## ClinCal® - Calibrator

**Serum Calibrator, lyophilised, for Vitamin C /**

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### Intended use:

ClinCal® Serum Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

### Reconstitution:

Add exactly 0.5 ml HPLC-water to the vial and mix for max. 3 min. After that the lyophilisate is reconstituted and ready to use.

**Caution: Immediately after reconstitution the serum calibrator has to be prepared according to the instruction manual of the ClinRep® Complete Kit for Vitamin C and analysed.**

### Storage and stability:

This product will be stable until the expiration date when stored unopened below -18 °C in the dark. After reconstitution the serum calibrator has to be used immediately and cannot be stored. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 0.87 %.

### Pack size:

ClinCal® Serum Calibrator for Vitamin C: 5 x 0.5 ml, **order no.: 28013**

### Caution:

The human serum which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the serum calibrator should be considered as potentially infectious and treated with appropriate care.

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