ClinCal[®] - Calibrator

Plasma Calibrator, lyophilised, for β -Carotene /

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Intended use:

ClinCal[®] Plasma Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 0.5 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

This product will be stable until the expiration date when stored unopened below -18 °C in the dark. After reconstitution the stability of the analyte is, when stored in the dark:

- 1 day at 15 - 30 °C
- 7 days at 2 - 8 °C
- 90 days at below -18 °C (avoid repeated freezing and thawing)

The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 0.67 %.

Pack size:

ClinCal[®] Plasma Calibrator for β -Carotene: 5 x 0.5 ml, **order no.: 30013**

Caution:

The human plasma which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen. Nevertheless, the plasma calibrator should be considered as potentially infectious and treated with appropriate care.

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