ClinCal® - Calibrator

Serum Calibrator Set lyophilised / Serum-Kalibrator-Set lyophilisiert

FOR ANTIMYCOTICS / ANTIMYKOTIKA

REF	MS9613
LOT	1369
	2021-01

Intended use:

ClinCal® Serum Calibrators are used for calibration of the analysis-system. These lyophilised calibrators are based on human serum and are available as a calibrator set with 4 levels, including a blank. After reconstitution the calibrators have to be prepared like patient samples in one series of analyses.

After reconstitution the stability of the analytes is:

- 1 day when stored at 15 - 30 °C
- 7 days when stored at 2 - 8 °C
- 90 days when stored below -18 °C (avoid repeated freezing and thawing)

Mean values:

The mean values have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Reconstitution:

Add exactly 1.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Notes:

The concentrations of the analytes are chosen in a way that a linear calibration curve can be generated over a suitable range of concentration. According to quality assurance all ClinCal® Calibrators have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.10 %.

Pack size:

ClinCal® Serum Calibrator Set
Level 0 - 3
4 x 1 x 1 ml, **order no.: MS9613**

Precautions:

The human serum which was used for manufacturing the calibrators was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen, HIV- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the serum calibrators should be considered as potentially infectious and treated with appropriate care.

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2021-01

Level 0

This blank-level was manufactured from the same pool of human serum like the others, but it was not spiked with any Antimycotic Drug. /

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Method of Analysis / Analysenmethode: LC-MS/MS

Please note:

Amphotericin B, Anidulafungin and Caspofungin have been spiked additionally. For more information please contact customersupport@recipe.de.

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